

# Stool Collection Procedure and Labeling Method (Must-read)

## ✓ Stool Collection Procedure

Pull the stool-collecting stick of the cap slowly.


Obtain sample using then end of the stool-collecting stick by scraping or poking at the surface of the stool around 2-3 times.

In the case of diarrheal stool, make the diarrheal stool to whole of the stick.

Place the stool-collecting stick back into the container and tighten the cap by pushing it down.

Though a stock solution is in the container, return to our laboratory the stool-collecting container within 7days.

Store the stool-collecting container in a fridge in a sealed plastic bag if you can't hand it in straight away.

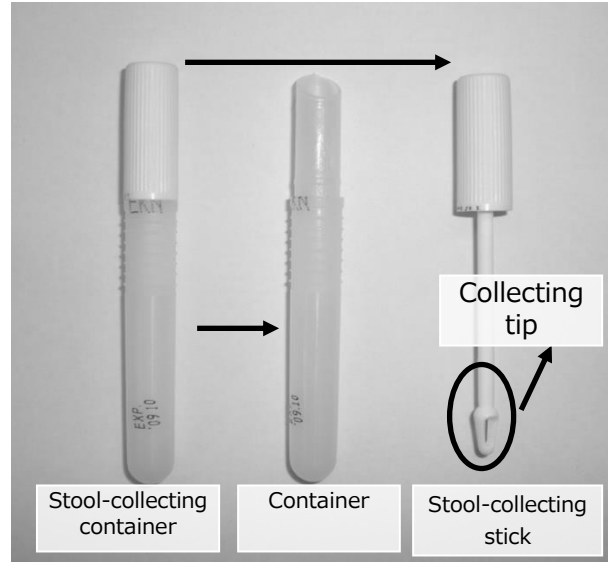

## ✓ Inspection-disabled

"Inspection-disabled" has a meaning as non-detect bacteria in a test result.

Quantity of sample of the stool is a very small amount, or the stool-collecting stick put only in the stored water of toilet bowl. Those acts are "Inspection-disabled".

Confirm that the stool attaches moderate amount to the tip of the stool-collecting stick.

## ✓ Personalized writing Label



Label the container with your name and the date.

Do not use the label of last month, Use the label of this month.

Do not dirty the bar code for prevention of data read error.

**Do not use a label of another person.**

## ✓ Non-entry Label



Label the container with your company name, job post name, and your full name. Write with a black ballpoint pen or a magic pen.